

# FULL PANEL. ZERO TOLERANCE.

NOW  
INCLUDING  
K3

The industry's first 18 panel drug testing cup with full adulteration panel.  
Designed to meet the needs of drug court, probation, and substance abuse professionals.



For Forensic Use Only

## ZERO TOLERANCE CUP

- Tests for 6-AM, Fentanyl, Tramadol, K2, K3 Synthetic Opiates (OXY) and ETG as well as other common drugs
- 18 drugs and 6 adulterants tested
- Low cutoffs (AMP 500 ng/mL), MET (500 ng/mL), and COC (150 ng/mL)
- Negative results as soon as 30 seconds
- Strong negative line intensity
- Clearly differentiated results at  $\pm 50\%$  cutoff

DRUGS TESTED		
Test	Description	Cutoff (ng/mL)
6-AM	Heroin	10
ETG	Alcohol Biomarker	500
FEN	Fentanyl	300
K2	Synthetic Cannabinoids	50
K3	Synthetic Cannabinoids	10
TRA	Tramadol	200
AMP	Amphetamine	500
BUP	Buprenorphine	10
BZO	Benzodiazepines	300
COC	Cocaine	150
MDMA	Ecstasy	500
MET	Methamphetamine	500
MOR	Morphine/Opiates	300
MTD	Methandone	300
OXY	Oxycodone	100
THC	Marijuana	50
PCP	Phencyclidine	25
BAR	Barbiturates	300
ADULTERANTS TESTED		
Oxidants		
pH		
Specific Gravity		
Glutaraldehyde		
Nitrates		
Creatinine		

### PROCEDURE (full directions for use)

1. Remove the cup from the sealed pouch.
2. Write the donor name or ID in the space provided.
3. Collect urine in the cup.
4. Read results after 5 minutes.

### INTERPRETATION OF RESULTS

**Negative Result:** If there is a red or pink line next to the test line under the drug name, the sample is considered negative for that drug. If a test line appears for all drugs, the sample is considered negative. Certain lines may appear lighter or thinner than other lines. No further testing is required with samples showing negative test results.

**Preliminary Positive Result:** If NO red or pink test line appears under the drug name, the sample may contain that drug. You should send the sample to a laboratory for additional testing.

**Invalid Result:** A control line should appear on every test strip. If you do not see a control line on any of test strips, your result is invalid.

### INNOVATIVE AND COMPETITIVELY-PRICED DRUG SCREENING PRODUCTS

# FACT<sup>LLC</sup>

Fast Accurate Cost-Effective Testing

When you need to know!

856.288.9049

For more info, email [smitnick@factllc.net](mailto:smitnick@factllc.net)