



## SMARTOX® PERSONAL BREATH TESTER

- One Touch Operation – Simply power on your SMARTOX® Personal Breath Tester and begin testing in seconds
- Optimal Breath Capture – Deep lung sample ensures most accurate reading
- DOT/NHTSA Compliant – Meets all federal requirements for a breath alcohol screening device

**European Fuel Cell Technology** – Advanced technology means more accurate results.

**Optimal Breath Capture** – Deep lung sample ensures most accurate reading.

**Test Counter** – Keeps track of administered tests so you never have to guess when it's time to recalibrate.

**One Touch Operation** – Simply power on your SMARTOX® Personal Breath Tester and begin testing in seconds.

**Color Changing LCD Screen** – BrAC results are displayed on a bright, easy-to-read LCD display. Color of display changes when reading is above 0.05% BrAC.

**DOT/NHTSA Compliant** – Meets all federal requirements for a breath alcohol screening device.

**Extra Mouthpieces** – Includes five mouthpieces, making it easy and sanitary to test your friends and family (additional mouthpieces available for purchase).

**Power** – Runs on AAA batteries (included) so your breath tester is ready when you are. No waiting on a device to charge.

**Warranty** – All SMARTOX® breathalyzers are backed by a full 90-day warranty.

### PROCEDURE

1. Wait at least 15 minutes after your last food or drink.
2. Place mouthpiece on the inlet.
3. Press and hold the POWER button until the unit powers on. Release once the unit is on.
4. Wait for the timer on the screen to countdown to zero.
5. Take a deep breath and exhale into the unit once the screen reads BLOW.
6. Blow until the unit clicks.
7. Your test result will be displayed on the screen and the color of the display will change if the result is above the alarm point.